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EN

Instructions for use for condoms made of natural rubber latex

Check the wrapper of the condom for integrity. If it is visibly damaged, discard the condom and take a new condom from a new, undamaged packaging. Cautiously tear the wrapper of the condom at the jagged edge. Treat the condom with caution upon removal from the packaging in order to avoid damages to the condom from fingernails, jewellery, etc. Slip the condom over the stiff penis before the first contact between the penis and the partner's body. To do so, completely pull back the foreskin and gently press the reservoir of the condom with thumb and index finger to remove any air. Then place the condom on the glans with the rolling edge to the outside and roll it over the whole penis. Do not re-use an improperly applied condom, since the purpose of the condom (having a supporting effect on the prevention of sexually transmitted infections and pregnancy) might no longer be ensured in the case of re-use. Please stop and re-check if you feel that the condom can slip. Excessive tension on the penis might result in a rupture in the condom. For this purpose, draw back the penis. Hold the condom at the root of the penis so that it does not slip off. Draw back the penis in the same manner soon after ejaculation, but at the latest before it starts to slacken. Only use the condom once. Re-use may result in an increased risk of failure of the condom or of infection. These condoms are made of natural rubber latex which might cause allergic reactions including anaphylactic shock in people with latex allergy. If the condom becomes leaky or bursts during

use, seek medical help as soon as possible, at most within 72 hours. If intolerances arise during the use of the condom, also seek medical advice. Additional lubricants can be used in order to remedy an insufficient lubricating effect. Please absolutely make sure that condom tolerability is displayed in the labelling of the lubricant. Please avoid the use of oily lubricants such as petroleum jelly, baby oil, body lotions, massage oils, butter, margarine, etc., because these impair the integrity of the condom. In the case of simultaneous use of drugs such as suppositories, ointments, creams, etc. which might come into contact with the condom, it is necessary to ask a doctor or pharmacist about their tolerability. Store condoms in the original packaging, in a cool and dry place protected from sunlight. Please observe the expiry date (see at EN). The condoms must not be used after the stated expiry date. Dispose of used condoms only with the household waste and do not throw them into the toilet. Condoms only offer their optimal protection in vaginal intercourse. No type of contraceptive guarantees 100% protection from pregnancy or infections.

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Donaxis GmbH
Amor Gummiwaren GmbH