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Brevet US n° 5 741 509

Brevet européen n° 1439866, 1409034

*Mustoe TA et al. Plast Reconstructive Surg 2002; 110:560-71

Advanced formula scar gel

Kelo-cote® is a lightweight, self-drying silicone gel for the treatment of scars. Kelo-cote® rapidly dries to form a sheet; this Silicone Gel Sheet layer is gas permeable, flexible and waterproof. Kelo-cote® forms a bond with the stratum corneum (the outer layer of dead skin cells) forming a protective barrier against chemical, physical and microbial invasion of the scar site while assisting with hydration. This creates an environment which allows the scar to mature through normalized collagen synthesis cycles, and improves the physiological and cosmetic appearance of the scar.

Kelo-cote® has been shown to flatten, soften and smooth scars, relieve the itching and discomfort of scars, as well as reduce the discoloration associated with scars.

Silicone gels have been shown to be effective in treating abnormal scarring in the form of hypertrophic scars and keloids. The 2002 International Clinical Recommendations on Scar Management recommended "silicone should be the first line of treatment in the initial management of scars and in the prevention of hypertrophic scars and keloids."

Kelo-cote® can be used on joints, flexures, hairy areas and large area scars such as those caused from burns or major trauma. Kelo-cote® can be used in conjunction with pressure garments or other scar therapy options.

Intended use:

Kelo-cote® Advanced Formula Scar Gel is intended for the management of old and new hypertrophic and keloid scars resulting from general surgical procedures, trauma, wounds and burns.

Kelo-cote® is suitable for use on children and people with sensitive skin.

Directions for use:

- Ensure that the affected area is clean and dry.
- Apply Kelo-cote® to the area as a very thin layer and allow to dry.
- Apply Kelo-cote® twice daily.
- For maximum effect, Kelo-cote® Gel should have 24 hour contact with the skin.
- Once dry, Kelo-cote® can be covered with pressure garments, sun block or cosmetics.

If Kelo-cote® has not dried within 4-5 minutes, you have probably used too much. Continue to remove the excess and allow the drying process to continue.

Recommended minimum treatment time is 60-90 days. Larger and older scars may take longer and continued use is recommended if improvement is still seen after the initial 90 days.

Tips for drying:

Kelo-cote® drying process is assisted by the temperature difference between the gel and the skin. To reduce drying time in hotter climates, keep Kelo-cote® in the refrigerator. In colder weather, use the low setting on a hair dryer to reduce drying time.

Caution:

Avoid direct contact with eyes, mucous membranes, third degree burns and open wounds. Kelo-cote® should not be used on dermatological conditions that disrupt the integrity of the skin. Kelo-cote® may stain clothing if it is not completely dry.

For external use only. If irritation occurs, discontinue use and consult your physician. Keep this product out of the reach of children. Store below 25°C. Do not use after the expiration (EXP) date printed on the tube.

Ingredients: Polysiloxanes, Silicon Dioxide

Contents: 6g, 15g & 60g

www.kelocote.co.uk

Manufacturer:

Device Authorised Representative, Alliance Pharma S.r.l., Viale Restelli 5, 20124 Milano, Italy

Advanced Bio-Technologies, Inc, 4830 West Kennedy Blvd, Suite 600, Tampa, Florida 33609, USA

US Patent 5,741,509

Eur. Patent-Nr. 1439866, 1409034

*Mustoe TA et al. Plast Reconstructive Surg 2002; 110:560-71

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