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### CONTRAINDICATIONS

- The wound gel must not be used:
  - in cases of known hypersensitivity to any of the ingredients,

**LAVANID® - Wound Gel / LAVANID® - Wound Gel V+**
**DESCRIPTION AND PRODUCT PROFILE**
The surface of chronic wounds is generally made up of devitalized and necrotic tissue, and is also colonised with bacteria. For wound treatment to be successful the wound surface needs to be removed, e.g. through surgical debridement. Wound irrigation and moistening are a necessary part of this process. Chronic wounds exhibit dysfunctional wound healing processes. This is why treatment with preparations that are very well tolerated by the tissues is indicated.

LAVANID® Wound Gel and LAVANID® Wound Gel V+ are sterile, ready-to-use wound gels for cleaning and moistening wounds. LAVANID® Wound Gel V+ is more viscous than LAVANID® Wound Gel. As preservative, Polyhexanide 0.04% is used as a preservative. Polyhexanide is evaluated as the agent of choice for chronic wounds and burns. In addition to cleaning and moistening wounds, this wound gel is also appropriate for preservative dressings, e.g. for wounds colonised with multiresistant bacteria. Preservative dressings reduce the risk of carrying the bacteria from the wound to the patient's environment and they protect the wound from bacteria getting in from the outside.

**COMPOSITION**
Ringer's solution (isotonic electrolyte solution made up of sodium chloride, potassium chloride, calcium chloride 2 H<sub>2</sub>O, water for injection), glycerin, macrogol, hydroxyethylcellulose, polyhexanide 0.04%.

**INDICATIONS**
The wound gel is used to moisten and clean wounds and to moisten bandages and wound dressings with preservative.

**TISSUE TOLERANCE**
Experimental and clinical studies have shown that the combination of Ringer's solution with 0.04% polyhexanide is very well tolerated by tissue. Clinical trials with this wound gel have shown that it does not negatively affect the formation of healthy new tissue. There are no local signs of irritant conditions. Use of this product is pain-free for patients.

Because it is so well tolerated by tissues the wound gel is suitable for repeated and long-term wound treatment.

**INSTRUCTIONS FOR USE**

- Prepare and clean the wound as usual.
- Apply the wound gel to the wound or a wound dressing (e.g. pads or compresses). Dress wound as usual. The wound gel should remain on the wound until the next dressing change. For optimal wound moistening we recommend increasing the amount of wound gel and lengthening the intervals between dressing changes. If the dressing sticks to the wound while being changed, moisten the dressing with a rinsing solution (e.g. LAVANID® Wound Rinsing Solution) before removing it.
- If the wound gel is going to be used more than once, please read the shelf-life information.

**DISPOSAL INFORMATION**
Dispose of unused products or waste material in accordance with the specific national regulations.

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