
# boso medisol

comfort

<b>DE</b>	<b>Inhalator</b>	
	Bedienungsanleitung.....	5
<b>EN</b>	<b>Nebuliser</b>	
	Instructions for use.....	23
<b>FR</b>	<b>Inhalateur</b>	
	Mode d'emploi.....	38
<b>IT</b>	<b>Inalatore</b>	
	Istruzioni per l'uso .....	54
<b>ES</b>	<b>Inhalador</b>	
	Instrucciones de uso .....	70

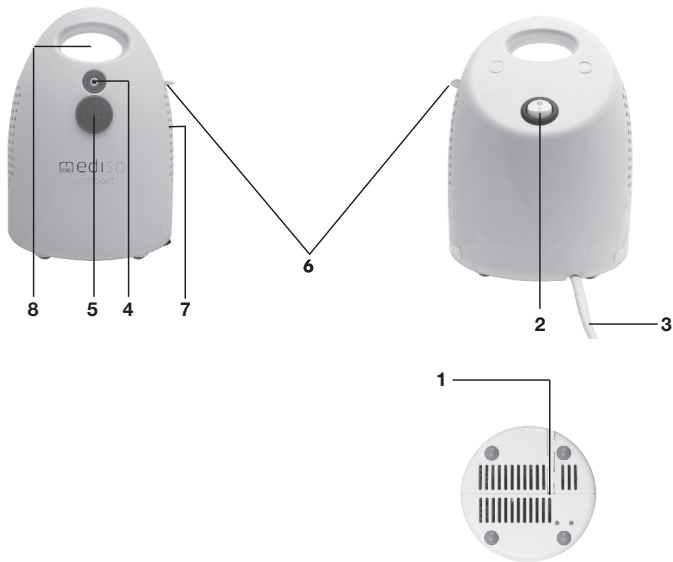**FIG. A / Abb. A**

9

10

11

12

13

14

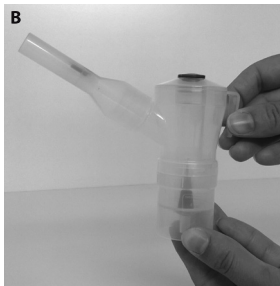
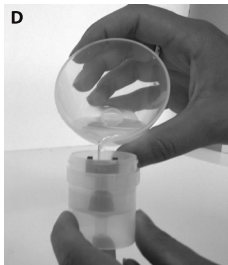
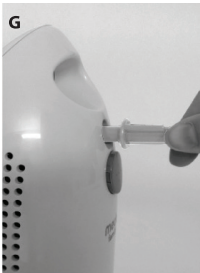
15

16

**B****C****D****E****F****G****H**

## Inhalt

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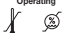
GERMANY.

## Contents

1. GETTING TO KNOW YOUR DEVICE .....	25
2. WARNINGS AND SAFETY NOTES .....	25
3. INSTRUCTIONS FOR USE .....	29
4. CLEANING AND MAINTENANCE .....	31
5. POSSIBLE PROBLEMS AND HOW TO SOLVE THEM .....	33
6. MAINTENANCE AND REPAIR .....	33
7. TECHNICAL SPECIFICATIONS .....	34
8. WARRANTY .....	37

## SYMBOLS ON THE DEVICE AND PACKAGING

	ON
	OFF
	TYPE BF
	CLASS II
	ALTERNATING CURRENT
	KEEP DRY
	TEMPERATURE

	ATTENTION
	SERIAL NUMBER
	MANUFACTURER
	READ THE INSTRUCTIONS FOR USE
	EUROPEAN COMMUNITY DIRECTIVE 93/42/EEC
	EUROPEAN AUTHORISED REPRESENTATIVE
	DISPOSE OF PACKAGING IN AN ENVIRONMENTALLY FRIENDLY MANNER
Storage / Transport 	PERMISSIBLE STORAGE AND TRANSPORT TEMPERATURE AND HUMIDITY
Operating 	PERMISSIBLE OPERATING TEMPERATURE AND HUMIDITY

# 1. GETTING TO KNOW YOUR DEVICE

## COMPONENTS - (PIC. A - page 2-3)

- |                       |                       |
|-----------------------|-----------------------|
| 1. Compressor housing | 9. Air tube           |
| 2. ON/OFF switch      | 10. Nebulizer         |
| 3. Power cord         | 11. Mouthpiece        |
| 4. Air outlet         | 12. Nosepiece         |
| 5. Filter case        | 13. Adult mask        |
| 6. Nebulizer holder   | 14. Child mask        |
| 7. Air-vent openings  | 15. Spare air filters |
| 8. Handle             | 16. Carrying case     |

## 2. WARNINGS AND SAFETY NOTES

boso medisol comfort is an aerosol therapy system indicated for home use.

### WARNINGS

1. Use the device only as described in this manual and therefore as an aerosoltherapy system, following the indications of your doctor. Any use different from the intended one is to be considered improper and hence dangerous; the manufacturer cannot be held liable for any damage caused by improper, incorrect and/or unreasonable use, or if the equipment is connected to electrical installations which do not comply with current safety regulations.
2. Keep this manual for future reference.
3. Do not operate the unit in presence of any anaesthetic mixture inflammable with oxygen or nitrogen protoxide.

4. The correct functioning of the equipment can be affected by electromagnetic interferences which exceed the limits indicated by the European standards in force. In case this device interferes with other electrical devices, move it and plug it to a different power socket.
5. In case of failure and/or malfunction, read the «POSSIBLE PROBLEMS AND HOW TO SOLVE THEM» section. Do not handle or open the compressor housing.
6. For repair operations address only to a technical service centre authorized by the manufacturer and require the use of original spare parts. The non-observation of the above mentioned indications can compromise the device safety.
7. Keep children away from packaging materials (risk of suffocation).
8. Comply with the safety regulations concerning the electrical devices and in particular:
  - use only original accessories and components.
  - never submerge the unit in water.
  - never wet the device, it is not protected against water penetration.
  - never touch the unit with wet or moist hands.
  - do not leave the unit exposed to the weather elements.
  - place the unit on a stable and horizontal surface during its operation.
  - this device is not intended for use by children or people with restricted physical, sensory (e.g. reduced sensitivity to pain, visual impairment) or mental skills or a lack of experience and/or lack of knowledge, unless they are supervised by a person who is responsible for their

safety or are instructed by such a person in how to use the device.

- do not pull the power cord or the device itself to unplug it from the power socket.
  - the power plug is the separation element from the grid power; keep the plug accessible when the device is in use.
9. Before plugging in the device, make sure that the electrical rating, shown on the rating plate on the bottom of the unit, corresponds to the mains rating.
  10. In case the power plug provided with the device does not fit your wall socket, address to qualified personnel for the replacement of the plug with a suitable one. In general, the use of adapters, simple or multiple, and/or extension cables is not recommended. If their use is indispensable, it is necessary to use types complying with safety regulations, paying however attention that they do not exceed the maximum power limits, indicated on adapters and extension cables.
  11. Do not leave the unit plugged in when not in use; unplug the device from the wall socket when it is not operated.
  12. The installation must be carried out according to the instructions of the manufacturer. An improper installation can cause damage to persons, animals or things, for which the manufacturer cannot be held responsible.
  13. The power cord of this device cannot be replaced by the user. In case of a power cord damage, address to a technical service centre authorized by the manufacturer for its replacement.
  14. The power supply cord should always be fully unwound in order to prevent dangerous overheating.

15. Before performing any maintenance or cleaning operation, turn off the device and disconnect the plug from the main supply.
16. Some parts of the unit are so small that they may be swallowed by children; keep the equipment out from children's reach.
17. Do not allow any objects to enter or obstruct the ventilation openings.
18. If you decide not to use the device any longer, it is recommended to dispose of it according to the current regulations.
19. Make sure to:
  - use this device only with medicines prescribed by your doctor.
  - make the treatment using only the accessory recommended by your doctor depending on the pathology.
  - use the nosepiece accessory only if expressly indicated by your doctor and paying attention to NEVER introduce the bifurcations in the nose, but only bringing them as close as possible.

### **Important**

Essential oils, cough syrups, gargling solutions and drops to be used as a rub or in a steam bath are wholly unsuitable for inhalation using a nebuliser. These additives are often viscous and can impair the correct functioning of the device and therefore the effectiveness of the application in the long term. For individuals with a hypersensitive bronchial system, medications containing essential oils may under certain conditions cause an acute bronchospasm (a sudden cramp-like restriction of the bronchi with shortness of breath). Consult your doctor or pharmacist in relation to this matter.

20. Check in the medicine package leaflet for possible contraindications for use with common aerosoltherapy systems.
21. To avoid strangulation and entanglement, keep cable and air tubes out of reach of young children.
22. Do not position the equipment so that it is difficult to operate the disconnection device.
23. For greater hygienic safety, we recommend you to avoid using the same accessories for more than one person.

### **3. INSTRUCTIONS FOR USE**

The device must be checked before each use, in order to detect possible functioning anomalies and/or damages due to transport and/or storage. During inhalation, sit upright and relaxed at a table and not in an armchair, in order to avoid compressing your respiratory airways and impairing the treatment effectiveness.

Ensure you are calm and relaxed when inhaling the medication and breathe slowly and deeply to ensure that the medication reaches right down to the fine, lower bronchi. Breathe out normally.

1. After unpacking the device, check it for visible damages or defects; pay particular attention to cracks in the plastic housing, which may expose electrical components. Check for accessories integrity.
2. Before using the device, proceed with the cleaning operations as described in the «CLEANING AND MAINTENANCE» section.
3. Open the nebulizer by turning counterclockwise the top (Pic. B).

4. Make sure that the medicine conduction cone is properly fitted on the air conduction cone inside the nebulizer (Pic. C).
5. Put the prescribed quantity of medicine into the nebulizer (Pic. D).
6. Close the nebulizer by turning clockwise the two parts, paying attention that they are thoroughly sealed (Pic. E).
7. Connect one end of the air tube to the nebulizer (Pic. F) and the other end to the air outlet (Pic. G) on the device.
8. Connect the required accessory part (mouthpiece, adult mask, child mask or nosepiece) to the nebulizer. If you need to use the mask, insert it directly onto the nebulizer (Pic. H).

**Note**

The most effective form of nebulisation is by using the mouthpiece. Nebulisation using a mask is only recommended if it is not possible to use a mouthpiece (e.g. for children who are not yet able to inhale using a mouthpiece). When using a mask to inhale, take care to ensure the mask fits well and the eyes are unobstructed.

9. Plug the device into the wall socket, make sure that the main supply corresponds to the electrical rating of the device.
10. To start the treatment, set the "I/O" switch into the "I" position.

**ATTENTION: this device is for intermittent use: 30 min ON / 30 min OFF.**

11. Inhale the aerosol solution using the prescribed accessory.
12. When the treatment has been completed, switch the unit off by setting the "I/O" into the "O" position and disconnect the plug from the wall socket.

13. Wash the nebulizer and its accessories as described in the «CLEANING AND DISINFECTION OF THE ACCESSORIES» section.
14. The device requires no calibration.
15. Making modifications to the device or accessories is not permitted.
16. Once it has been properly prepared, the device can be used again. Preparation involves replacing all the accessories, including the atomiser and air filter, and disinfecting the surface of the device using a standard disinfectant. Please note that all accessories should be replaced if the device is used by more than one person.

## 4. CLEANING AND MAINTENANCE

The device is maintenance-free.

### **ATTENTION:**

During cleaning operations, make sure the internal parts of the unit are not in contact with liquids and that the power plug is disconnected.

- The cleaning of the device must be carried out by using a soft and dry cloth and non-abrasive cleansers.

## **CLEANING AND DISINFECTION OF THE ACCESSORIES**

Follow carefully the cleaning and disinfection instructions of the accessories as they are very important for the device performances and the therapy success.

### **1. After each treatment:**

- Disassemble the nebulizer by turning counterclockwise the top and remove the medicine conduction cone.

- Wash the components of the disassembled nebulizer, the mouthpiece, the masks and nosepiece by using tap water; dip in boiling water for 5 minutes.
- Reassemble the nebulizer components and connect it to the air-outlet (Pic. G), switch the device on and let it work for 10-15 minutes.

**Do not boil nor autoclave the air tube.**

## **2. Disinfection:**

Cold disinfecting solutions can be used, instead of boiling water, according to manufacturer's instructions.

## **REPLACEMENT OF THE NEBULIZER**

The nebulizer must be replaced after a long period of inactivity, in case it shows deformations or breakings, or when the nebulizer nozzle is obstructed by dry medicine, dust, etc. bosco recommends to replace the nebulizer after a period between 6 months and 1 year depending on the usage.

Use original nebulizers only.

## **REPLACEMENT OF THE AIR FILTER**

- Under normal usage conditions the air filter must be replaced after approx. 500 operating hours or 12 months at the latest.
- bosco recommends to periodically check the air filter (10 – 12 treatments) and, if the filter shows a grey or brown colour or is wet, replace it. Extract the filter (Pic. A-5) and replace it with a new one.

- Do not try to clean the filter for reusing it. The air filter shall not be serviced or maintained while in use with a patient.

**Use original filters only. Do not use the device without filter.**

## **5. POSSIBLE PROBLEMS AND HOW TO SOLVE THEM**

Contact an authorized Customer Service Centre in case you need help about the use and the maintenance of the equipment.

### **Problems:**

1. The device does not switch on.
2. The device does not nebulize or nebulizes weakly.

### **Solutions:**

1. Make sure the power plug is firmly fitted to the wall socket.
2. Make sure that the device has been operating within operating limits indicated in this manual (30 min ON / 30 min OFF).
3. Make sure that the ends of the air tube are tightly fitted to the main unit and nebulizer.
4. Verify that the nebulizer is not empty or has been filled with the proper quantity of medicine (max 8 ml). Verify that the nebulizer nozzle is not obstructed.

If the device is not able to function properly again, please contact boso Customer Services (see 8. Warranty).

## **6. MAINTENANCE AND REPAIR**

In case of damage, please contact boso Customer Services (see 8. Warranty). Never open the device yourself. The unit has no

user-serviceable parts within and does not need internal maintenance or lubrication.

## 7. TECHNICAL SPECIFICATIONS

Model:	boso medisol comfort
Power supply:	230 V; 50 Hz AC
Minimum filling volume:	2 ml
Maximum filling volume:	8 ml
Nebulization rate:	0.37 ml/min
Maximum pressure:	2.2 bar
Weight:	1.5 kg
Size:	188 mm × Ø 136 mm
Noise level:	52 dBA
Expected service life:	1000 hour
Material:	Atomiser (PP), mouthpiece (PP), nosepiece (PP), masks (PP+TPR)
Residual volume:	approx. 0.7 ml

- Class II device as regards protection against electric shocks.
- Nebulizer, mouthpiece and masks are type BF applied parts.
- Device not protected against sprinkles.
- Device not suitable for use in presence of anaesthetic mixtures inflammable with air, oxygen or nitrogen protoxide.
- Device for intermittent use (30 min ON / 30 min OFF).
- Device not suitable for use in anaesthesia or lung ventilation systems.

**NOTE:** The technical specifications may change without prior notice.

### **Applied standards**

This device and its accessories comply with European standards EN60601-1 and EN60601-1-2 (CISPR 11, IEC61000-3-2, IEC61000-3-3, IEC61000-4-2, IEC61000-4-3, IEC61000-4-4, IEC61000-4-5, IEC61000-4-6, IEC61000-4-7, IEC61000-4-8, IEC61000-4-11), as well as EN13544-1, and is subject to special safety measures in terms of electromagnetic tolerance. The unit conforms to the requirements of the European Directive for Medical Products 93/42/EEC, the MPG (German Medical units Act).

### **Notes on electromagnetic compatibility**

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the

device's electromagnetic immunity; this can result in faulty operation.

## ENVIRONMENT CONDITIONS

### Storage

Temperature: MIN -25 °C - MAX +70 °C  
Humidity: MIN 10% RH - MAX 95% RH  
Atmospheric pressure: 700 hPa – 1060 hPa

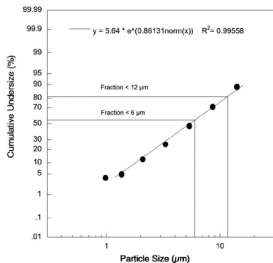
### Operation

Temperature: MIN +10 °C - MAX +40 °C  
Humidity: MIN 10% RH - MAX 95% RH  
Atmospheric pressure: 700 hPa – 1060 hPa

## AEROSOL CHARACTERISTICS IN ACCORDANCE WITH REGULATION EN 13544-1 APPENDIX CC

Aerosol output: 0.24 ml  
Aerosol output rate: 0.09 ml/min  
Particle size (MMAD): 5.64 µm

### Particle size diagram

## **8. WARRANTY**

We provide a 5-year manufacturer's warranty for this product from the date of purchase. The date of purchase must be proven by the receipt. Defects as a result of material or production faults will be remedied for free within the warranty period. The warranty claim does not extend the warranty period for the entire device, but only for the components that have been replaced. The warranty does not apply to wear and tear, transport damage and all damage resulting from improper handling (e.g. non-observance of the instructions for use) or caused by tampering by unauthorised persons. This warranty does not justify claims for damage against us. The purchaser's statutory claims for defects in accordance with Art. 437 of the German Civil Code (BGB) are not restricted.

In the event of a warranty claim, the device must be sent together with the original proof of purchase to:

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Distributed by: Bosch + Sohn GmbH u. Co. KG
Donawa Lifescience Consulting Srl
Globalcare Medical Technology Co., Ltd.